

## Profile Y-FLOW

- **Product Name** Y-FLOW
- **Classification** Medical Device(Therapeutic Vaginal Douche Apparatus)
- **Purpose of Use** A disposable device consisting of a nozzle and a container holding a cleansing solution (purified water), designed for insertion into the vagina to perform internal cleansing.
- **Raw Materials** Hyaluronic Acid, Lactic Acid, Polyvinylpyrrolidone
- **Packaging Unit** 7 EA / 1 BOX
- **Storage Method** Store at Room Temperature (1~30°C)
- **Manufacturing Requester** DSH Medical



## How to use

- 1 After washing your hands thoroughly, Open the top of the packaging, Take out the product, and remove the cap.
- 2 Hold the body of the product with your thumb and middle finger, place your index finger on the pusher, and insert it about 5-8cm from the injection port.
- 3 Lightly push the pusher all the way into inject all of the solution in the device at once.
- 4 During daily life, we recommend wearing a pantyliner as the solution may leak out, and we recommend using it before going to bed (recommended use 3-5 times a week).

# Y-FLOW

Perineal Protective Gel Dressing



## Y-FLOW Development Background

**7 out of 10 women experience vaginitis.**  
Vaginitis is like a cold in women,  
**So treatment and preventive care are essential.**

### Recommended for use



If you have symptoms of vaginitis



Vulvodynia, Itching



When supplementary treatment is required when taking vaginal tablets due to gynecological disease



If undergoing treatment for HPV test



When vaginal tablets are restricted due to chemotherapy, etc.

### Treat with medical devices approved by the Ministry of Food and Drug Safety

Caused by social and environmental factors, lifestyle habits, stress, etc.  
Changes in vaginal discharge, unpleasant odor, or discomfort.

A sudden feeling of discomfort before or after sexual intercourse, after using a bathhouse, sauna, or swimming pool, or during outdoor activities.



### Use a Y-FLOW for vaginal cleanliness care.

#### It's easy outside

Lightly wipe the flowing gel with a tissue or wet tissue without having to wash it with water.

- Use during daily activities (After about 2-4 hours)
- Use before going to sleep (Use it the next morning)

## Y-FLOW Features

- 1 Contains safe and effective ingredients that can **reduce inflammation and pain in the vaginal mucosa (hyaluronic acid, lactic acid, polyvinylpyrrolidone)**
- 2 **Medical device approved by the Ministry of Food and Drug Safety** whose effectiveness and safety have been verified through vaginal mucosa irritation tests
- 3 Contains **a large amount of lactic acid**, which maintains the pH level in the vagina, helping to **restore the self-cleaning effect** by proliferating beneficial bacteria and suppressing harmful bacteria.

### Why do I feel this discomfort?

This is because the pH balance in the vagina is broken.

In a woman's vagina, lactic acid bacteria and harmful bacteria coexist. When the beneficial bacteria, lactic acid bacteria, decrease and the harmful bacteria increase, a pH imbalance occurs in the vagina.

### Y-FLOW helps maintain **optimal pH balance** in the vagina.

Y-FLOW is composed of a slightly acidic product to suppress the growth of harmful bacteria and maintains optimal pH balance.

